
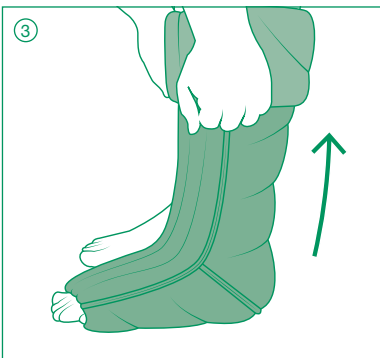
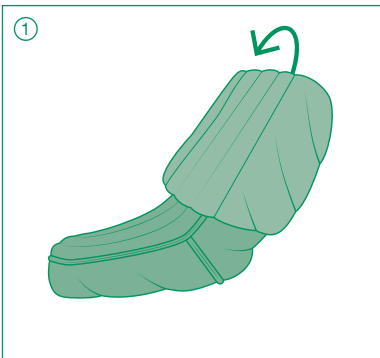
# Caresia™

Lower extremity foam compression bandage liner  
Unterzugmaterial aus Schaumstoff für die untere Extremität  
Matière de rembourrage en mousse pour extrémités inférieures  
Media interna de espuma para extremidad inferior

## Foot/Below Knee
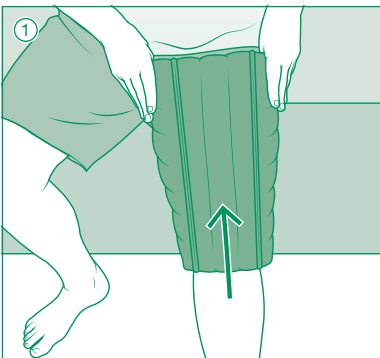
## Thigh


## Thigh

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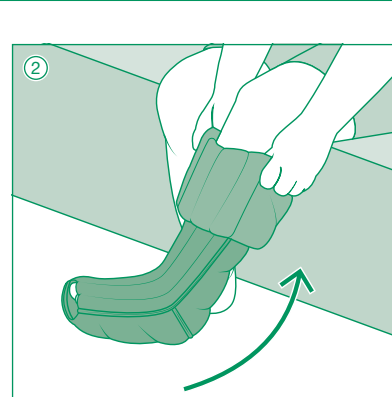## en Instructions for use

Thank you for choosing Caresia™!

Caresia Foot, Below Knee, and Thigh garments are off-the-shelf foam bandage liners for lymphedema and edema management of the foot and/or leg. Caresia can be used day or night in the intensive phase of therapy or in the self-management phase as recommended by your medical professional.

Please consult a trained medical professional prior to first use.

## Indications and patient groups

- Various stages of lymphedema
- Edema

## Intended uses

- Assists in softening of indurated/fibrotic tissue (when used with compression bandages/ compression garments)
- Fill concavities and/or pad bony prominences (when used with compression bandages/ compression garments)
- Diffuse pressure points (when used with compression bandages/ compression garments)

## Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Untreated congestive heart failure
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components

## Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

## Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

## Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

## Donning instructions

## Foot/Below Knee

If pairing a Below Knee and Thigh garment, apply Thigh garment first.

- Fold the upper portion of the garment inside out overlapping the lower portion.

- Slide the folded garment over the foot and heel.

- Unfold the garment onto your leg. Adjust for fit and comfort.

- If pairing a Below Knee and Thigh garment, unfold the Below Knee garment to overlap the knee portion of the Thigh garment.

- Apply short stretch bandages over top as directed by your medical professional.

## Thigh

- Slide the garment over your foot onto your knee and thigh ensuring the outer seams are at the front of the garment on both sides of your leg.

- Apply short stretch bandages over top as directed by your medical professional.

## Care instructions

L&R recommends all Caresia pads be washed and dried before the first extended use.

Caresia pads can be washed and dried with your other laundry.

- Machine wash with mild detergent on permanent press
- Do not use fabric softener, bleach, or other additives
- Do not dry clean
- Tumble dry on permanent press
- Do not iron

## Material content

Polyurethane, nylon (polyamide), spandex (elastane)  
Not mixed with natural rubber latex.

## Warranty

We guarantee your Caresia garment to be free of material and manufacturing defects during the recommended six month life of the garment.

## Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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**Lohmann & Rauscher AB**