en Instructions for use for Exo™ Upper extremity knit compression garment

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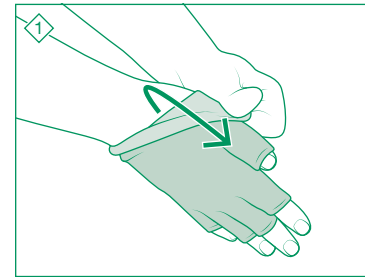
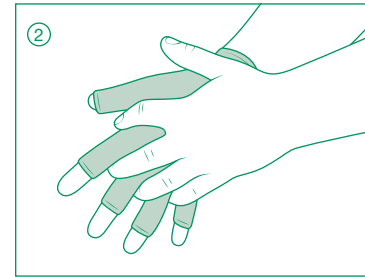
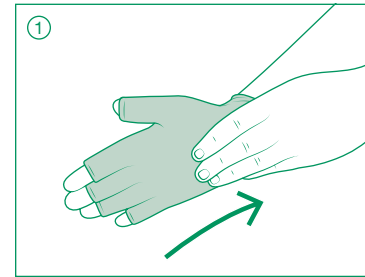
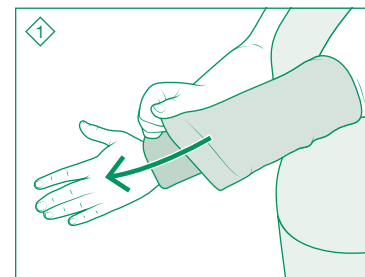
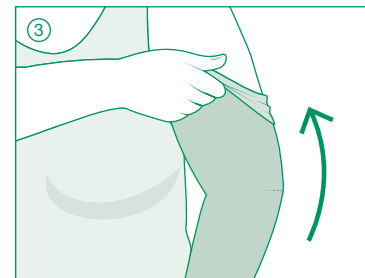
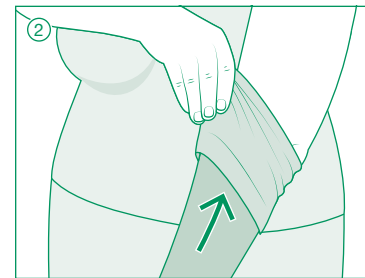
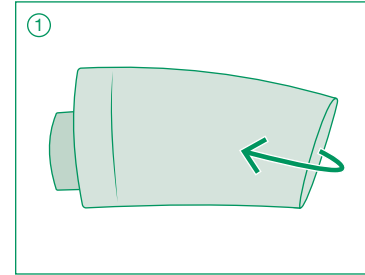
en | Instructions for use

Thank you for choosing Exo™!

Exo™ is a range of off-the-shelf or custom knit compression garments for the management of edema and lymphedema. Exo is designed to be used during the day and high activity.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Edema/swelling

Intended uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components

Precautions

- Cognitive impairment
- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

General guidance

- Do not stick fingernails into the knitted material when pulling it on and off.
- Do not wear jewelry while donning and doffing.
- Can use rubber gloves to help don and doff.
- Compression garments are easiest to put on when you first get out of bed in the morning.
- Do not overstretch the material.

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Donning and Doffing instructions

Glove/Gauntlet

- If pairing with an Arm Sleeve, don Arm Sleeve garment first.

Donning

- Slide your hand into the Glove/Gauntlet.
- Adjust for fit and comfort, ensuring the garment fits snugly into the finger web space(s).

Doffing

- To remove, gently pull the garment off inside out.

Arm Sleeve

Donning

- Fold the upper portion of the garment inside out overlapping the lower portion.
- Slide your hand into the folded garment and pull it up to your elbow.
 - ExoStrong garments have a seam which should be aligned to the outside of your wrist.
- Unfold the garment gently over your elbow and pull the rest of the way up ensuring it lays smoothly without creases.

Doffing

- To remove, fold the upper portion of the garment inside out and continue pulling it gently off your arm.

Care instructions

- To remove, fold the upper portion of the garment inside out and continue pulling it gently off your arm.

Washing

- Machine wash with mild detergent on permanent press
- Do not use fabric softener, bleach, or other additives
- Do not dry clean

Drying

- Tumble dry on permanent press
- Do not iron

Material content

Refer to garment tag for the composition of your specific garment. Not made with natural rubber latex.

Warranty

We guarantee your Exo garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

Material content

Refer to garment tag for the composition of your specific garment. Not made with natural rubber latex.

Warranty

We guarantee your Exo garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

Material content

Refer to garment tag for the composition of your specific garment. Not made with natural rubber latex.

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